



Distributed by Medusoft Pty. Ltd.  
www.medusoft.com.au  
Email: contact@medusoft.com.au

The portable salt-cave.  
The joy of breathing easy.  
The natural way of giving your  
respiratory system a boost.

Just a few minutes or regular  
(10-15 minutes) daily use can be  
of major benefit not only

- ✓ to asthma
- ✓ hay fever, allergy sufferers,
- ✓ smokers and snorers

but can also help overcome

- ✓ tuffiness,
- ✓ persistent coughing
- ✓ shortness of breath,
- ✓ wheeziness during colds,
- ✓ neutralize the daily damaging effects of air pollution,  
dust, smog, fog and secondary cigarette smoke.

Also beneficial to clean the lung for better performance

- ✓ for sporting activity
- ✓ singing performances playing
- ✓ wind-instruments ( like trumpet...)

For centuries salts have been used as folk remedies by  
many different cultures in regions around the world. Today  
people suffering from respiratory disorders travel to  
curative mines to inhale the salt brine vapour have  
brought amazing natural respiratory relief and treatment.

Now you can achieve the benefits of rock salt vapours  
easily and simply and in the comfort of your own home or  
on the road by using the compact Saltpipe.



Retailer:

## How does it work?

The Saltpipe inhaler filled with Miocene Mineral Halite Salt Crystals from the Transylvanian Praid Salt mine. Its salty therapeutic microclimate calms and cleanses the cells of the respiratory system. When using the Saltpipe the moisture of the passing air absorbs the micron sized particles of the salt that penetrates into the respiratory system to induce natural self cleansing mechanisms that flush away the impurities from the surface of the cells, mechanically cleanse the passage ways, help clear out the nasal cavities and help calm, heal and repair inflamed lungs and airways. Acting as a 'muco kinetic' and it helps restore the normal transport of mucus and unclog blockages in the bronchi and bronchioles..

## How to use it ?

You do not need to pump or press any button , just inhale through your mouth and exhale through your nose and you do not need to hold your breath as you would with the spray inhaler. When using the Saltpipe for the first time, please do not take a deep forced breath because the concentrated salt air in the Saltpipe will cause choking sensation. You may experience a salty taste at first but this will be lessen with use as the salt crystals vaporize within the pipe After using the Saltpipe for a couple of times, you may experience more coughing or wanting to spit, please do not let this put you off , this is the process of cleaning the breathing system. Regular use of the Saltpipe will cause these experiences to disappear resulting in easier breathing , less coughing, less soreness in the throat and less phlegm in the lung.

In order to achieve the best result the device should be used regularly. You may experience dry lips. If you have this problem, you may choose to apply lip-balm before use.

**Take care not to breath back into the pipe and please keep the Saltpipe in a dry place to prevent damage of salt by humidity.**

Test have shown that the quantity of salt inhaled into the body per normal use of 20 minutes per day is less than a microgram.

Disclaimer : The Saltpipe is a natural device and it can be used complimentary to current medication without side effects. No product or advice is solely intended to replace current medication. If you are concerned about the applicability relating to your conditions please consult your doctor.